



Attendant®


CONNECTED VITAL SIGNS MONITOR CORD MANAGEMENT STAND

Owner's Manual


Please keep and refer to this Owner's Manual.

Thank you for purchasing an Attendant® Connected Vital Signs Monitor Cord Management Stand from Direct Supply Equipment & Furnishings®, a division of Direct Supply, Inc. Please read this entire manual carefully and keep it for future reference. This manual will provide you with instructions, warnings, warranty information and other important information about your device. Share this information with your housekeeping, nursing and maintenance staff to help ensure the device is cared for properly.


Unpacking

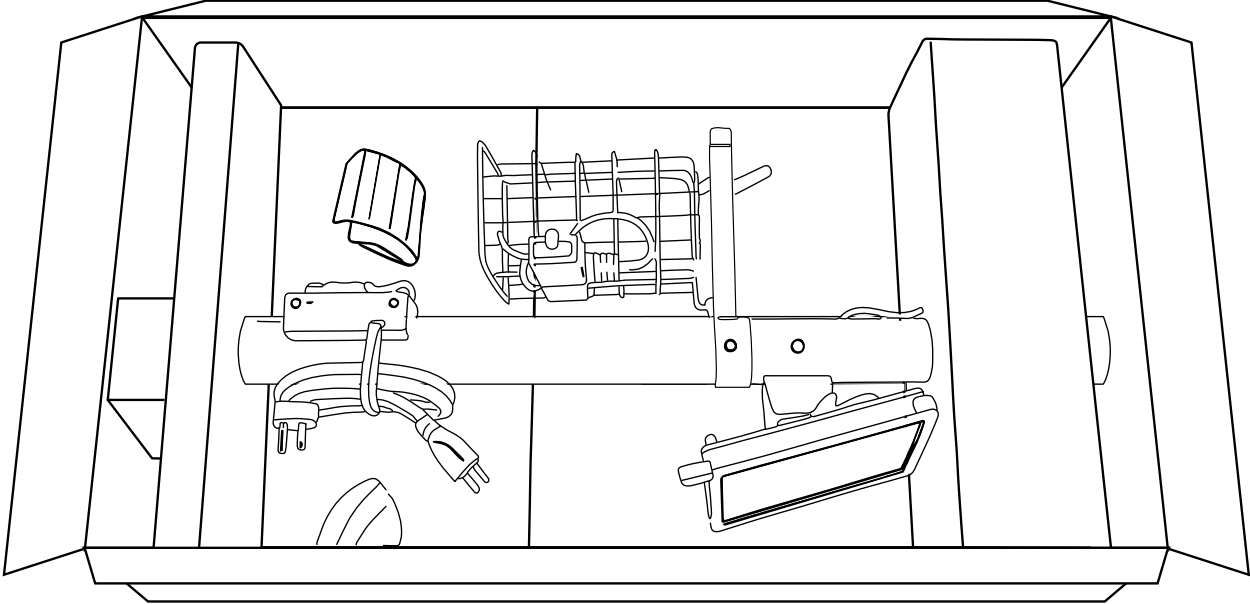

Parts List


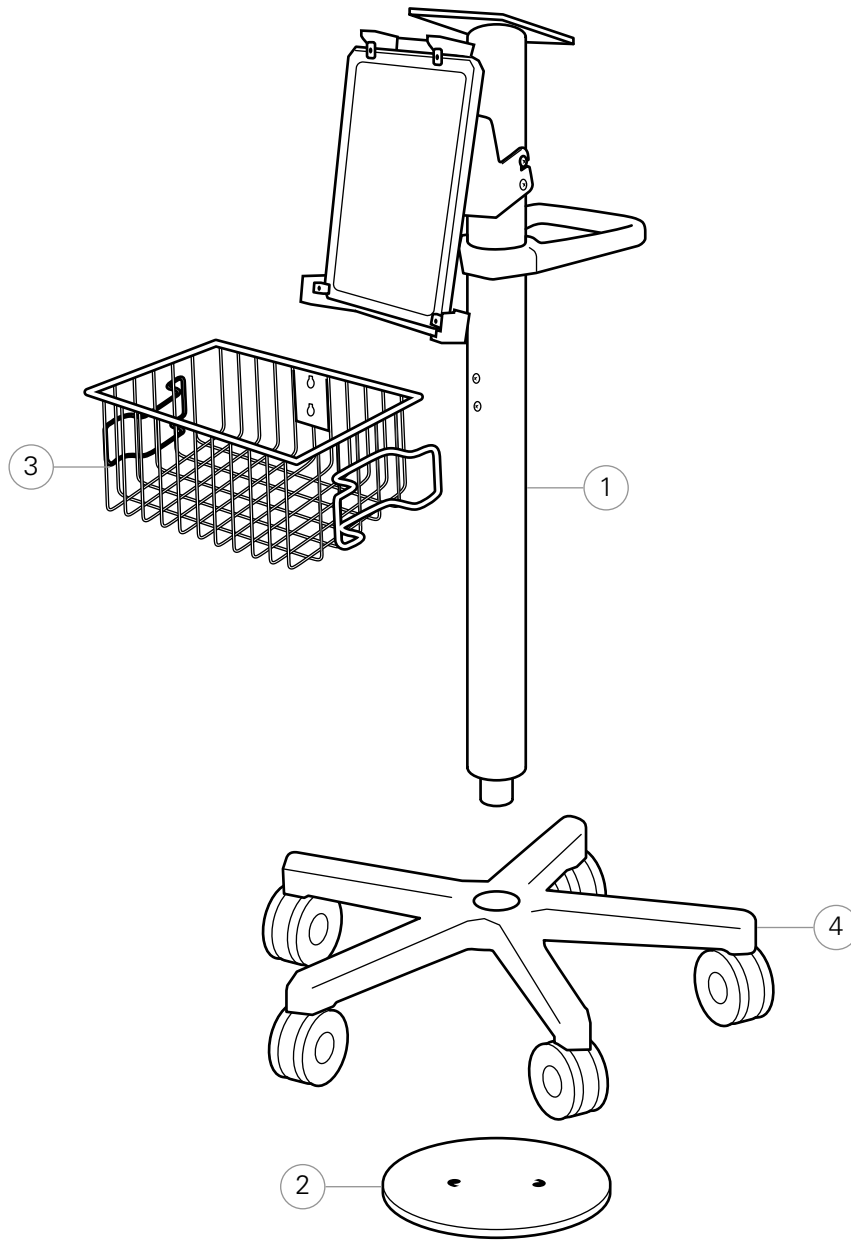


Item #	Description
1	Pole
2	Counterweight
3	Wire Basket
4	Black 5-Spoke Base
5	#2 x 1½" Phillips Magnetic Tip Screwdriver
6	Combination Wrench 10 mm



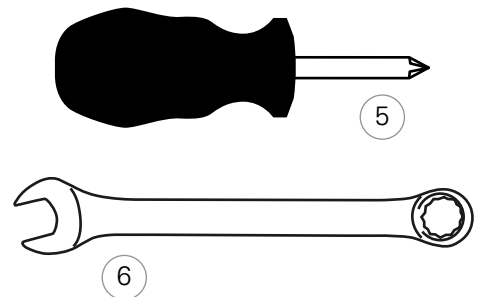


Assembly Instructions

Attaching the pole to the caster base

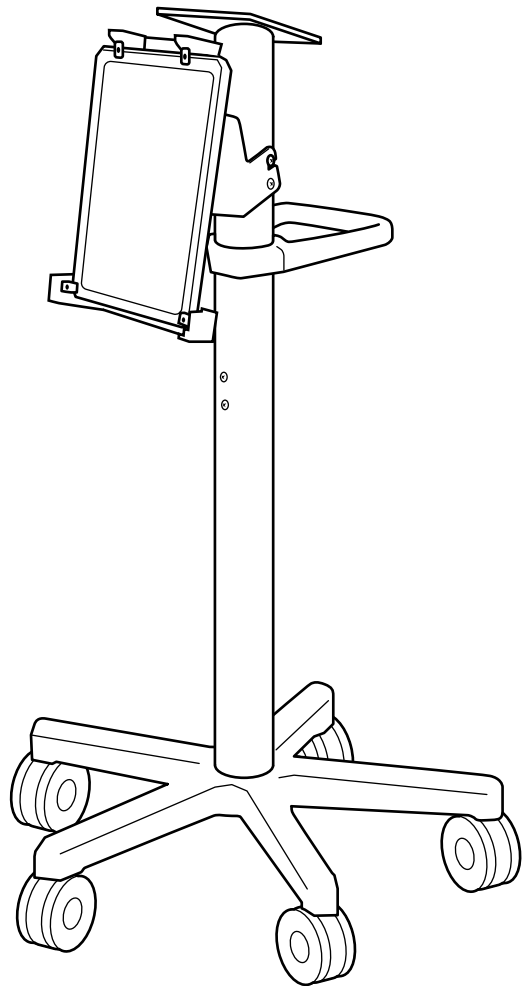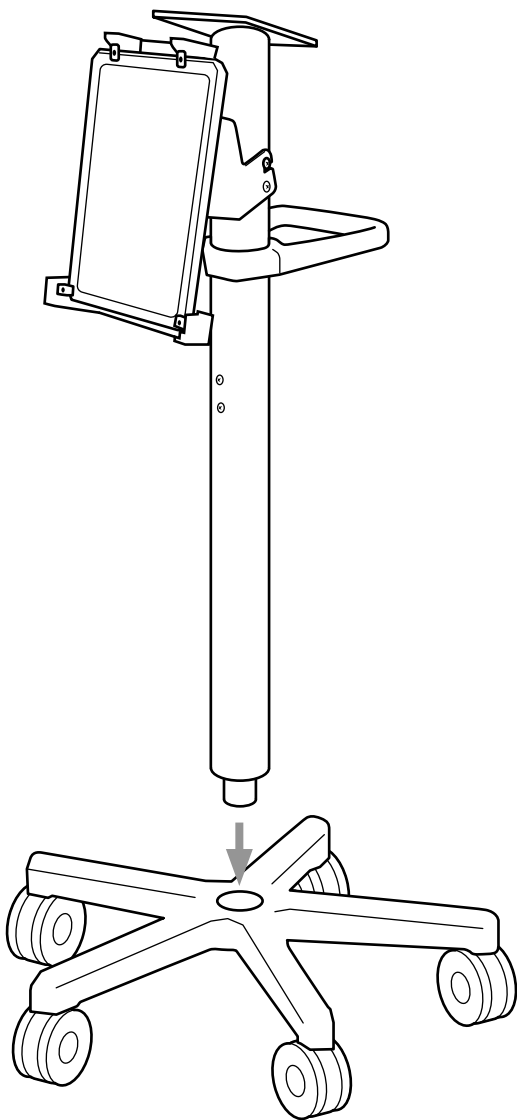




Attaching the counterweight

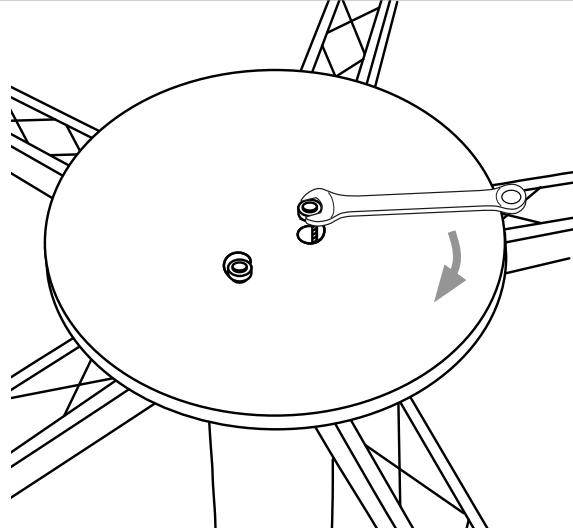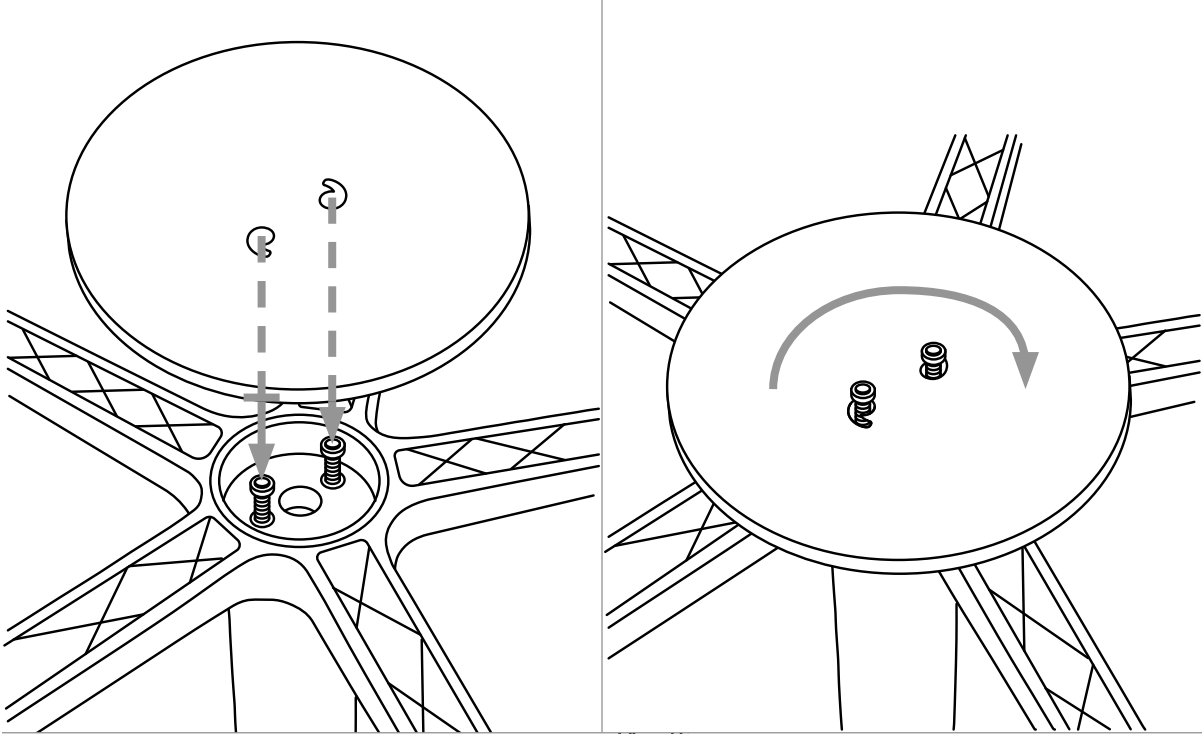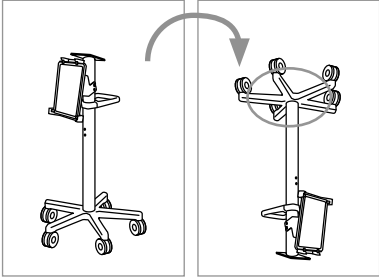



Ensure both nuts are fully tightened

Assembly Instructions *(cont.)*

Attaching the basket

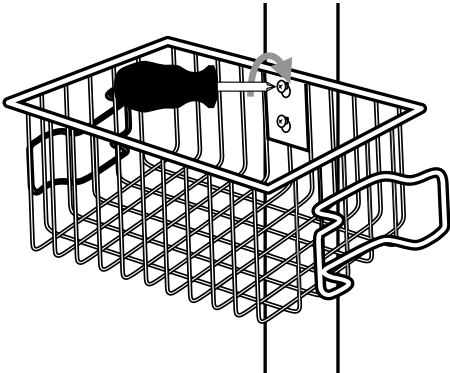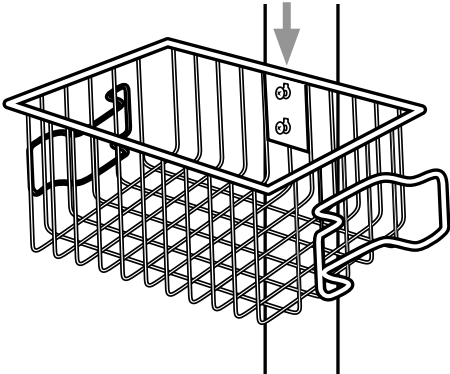
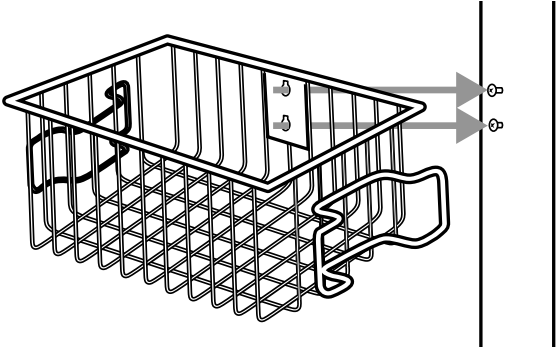


Attaching the VSM

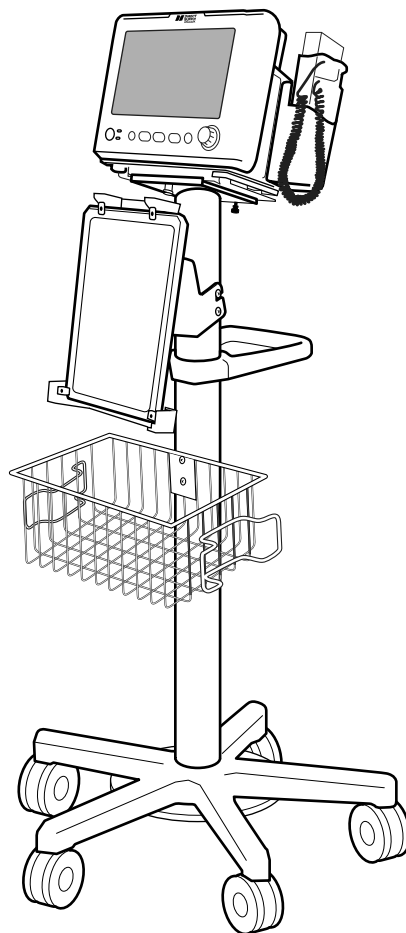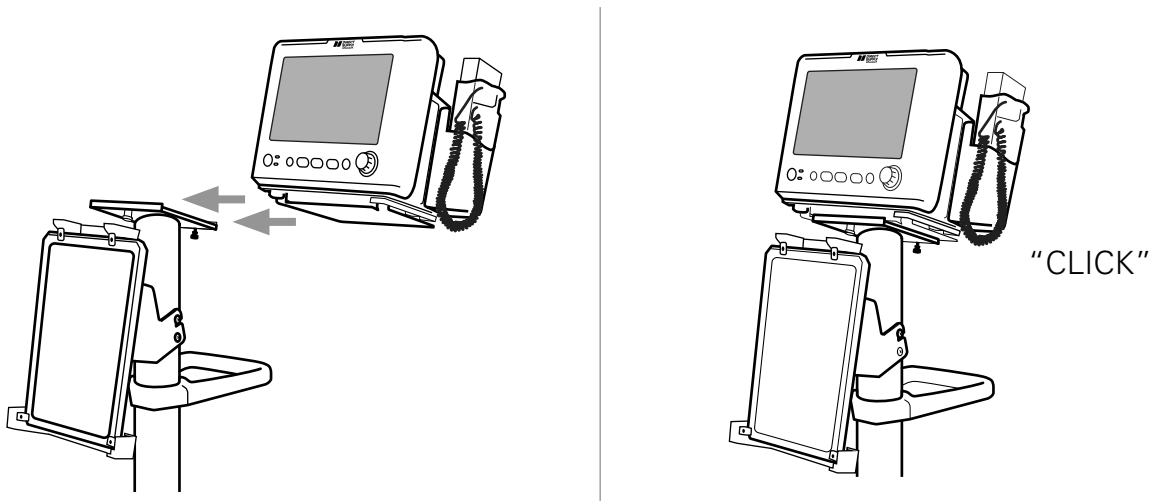




Cord Management

Guiding the VSM power cord through the pole

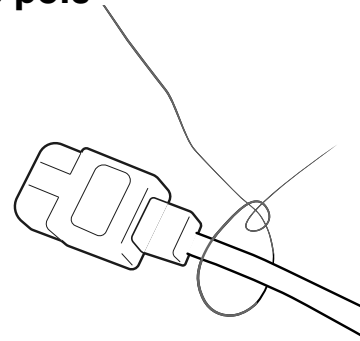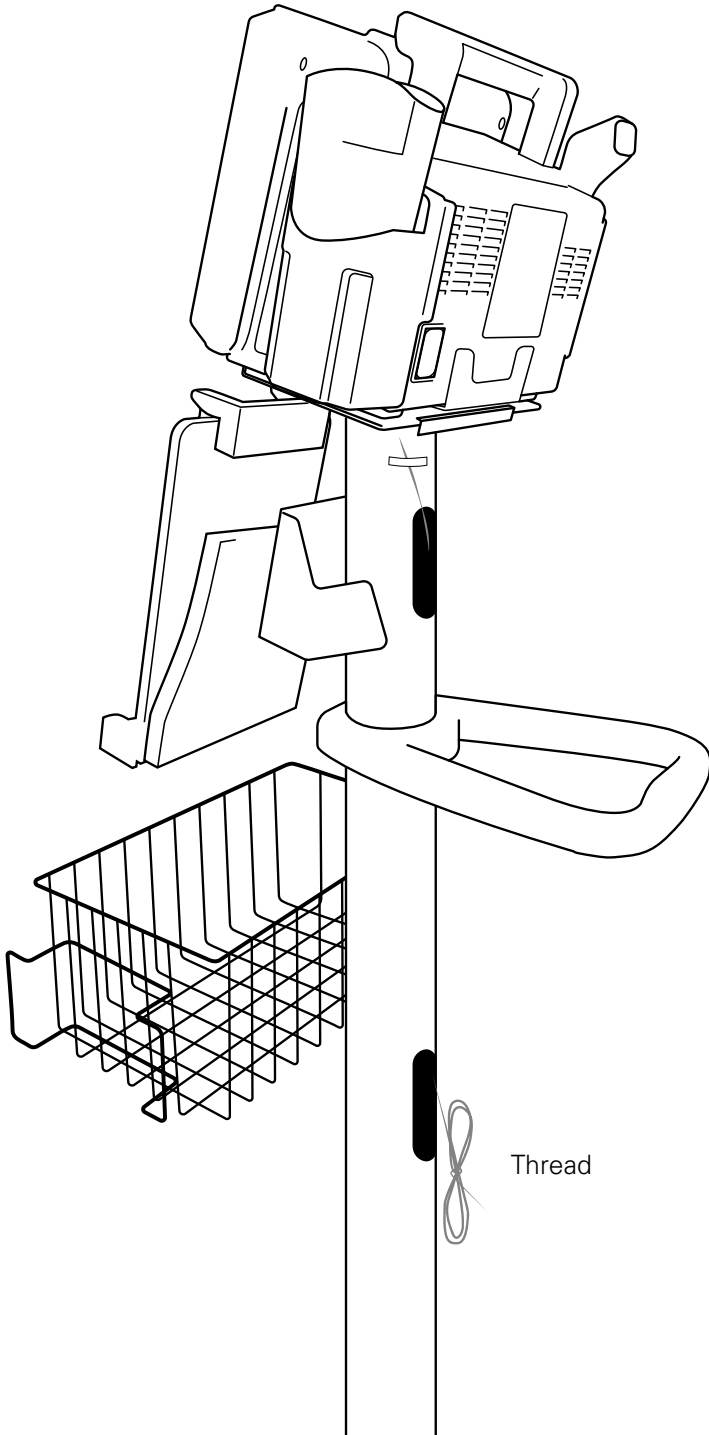



Tie string around Vital Signs Monitor plug

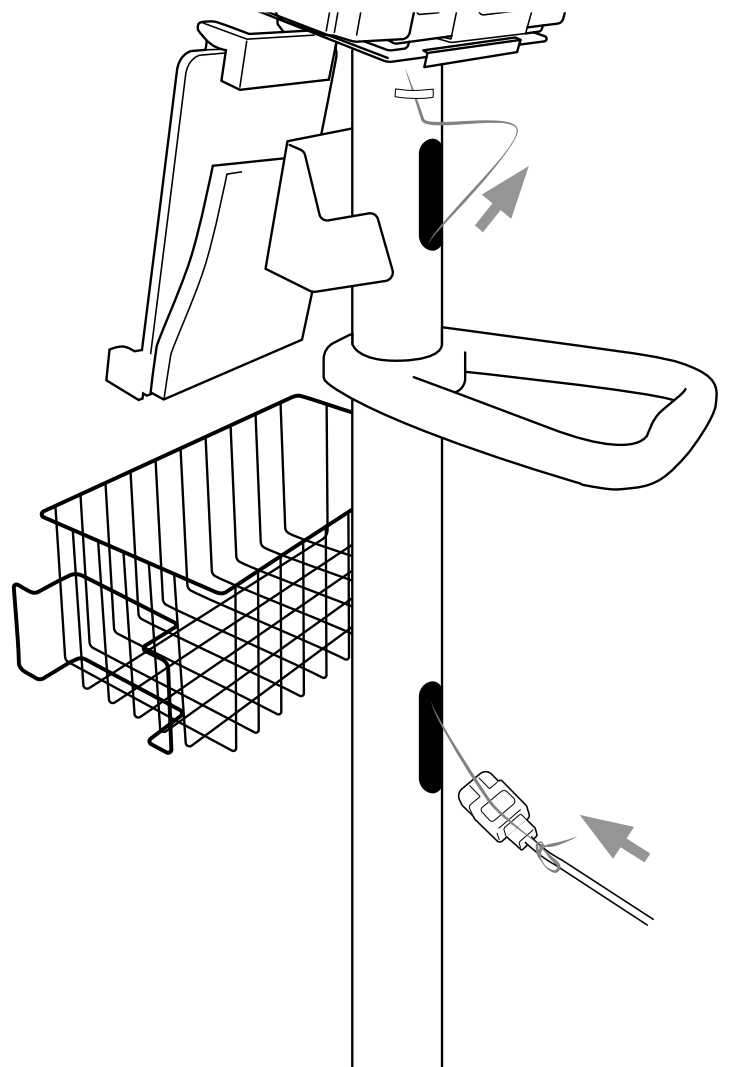


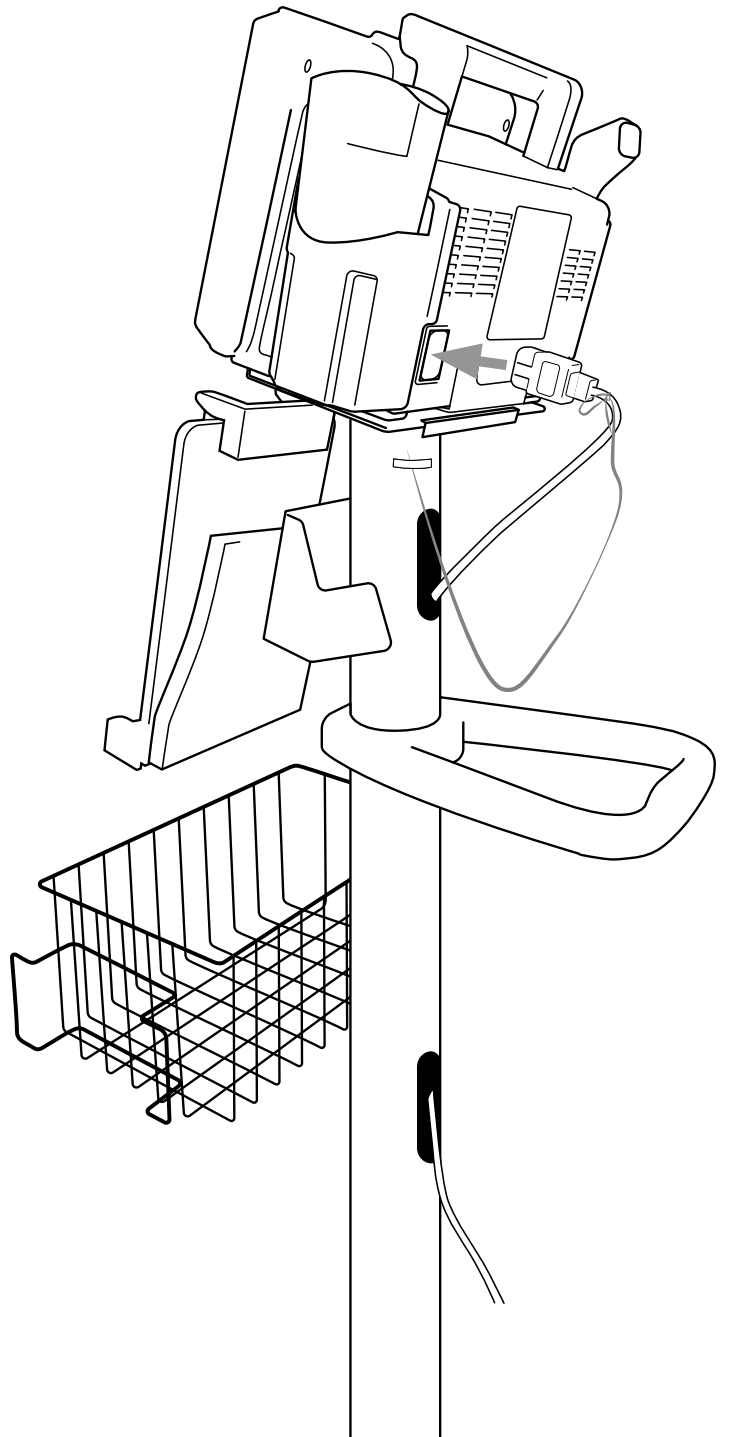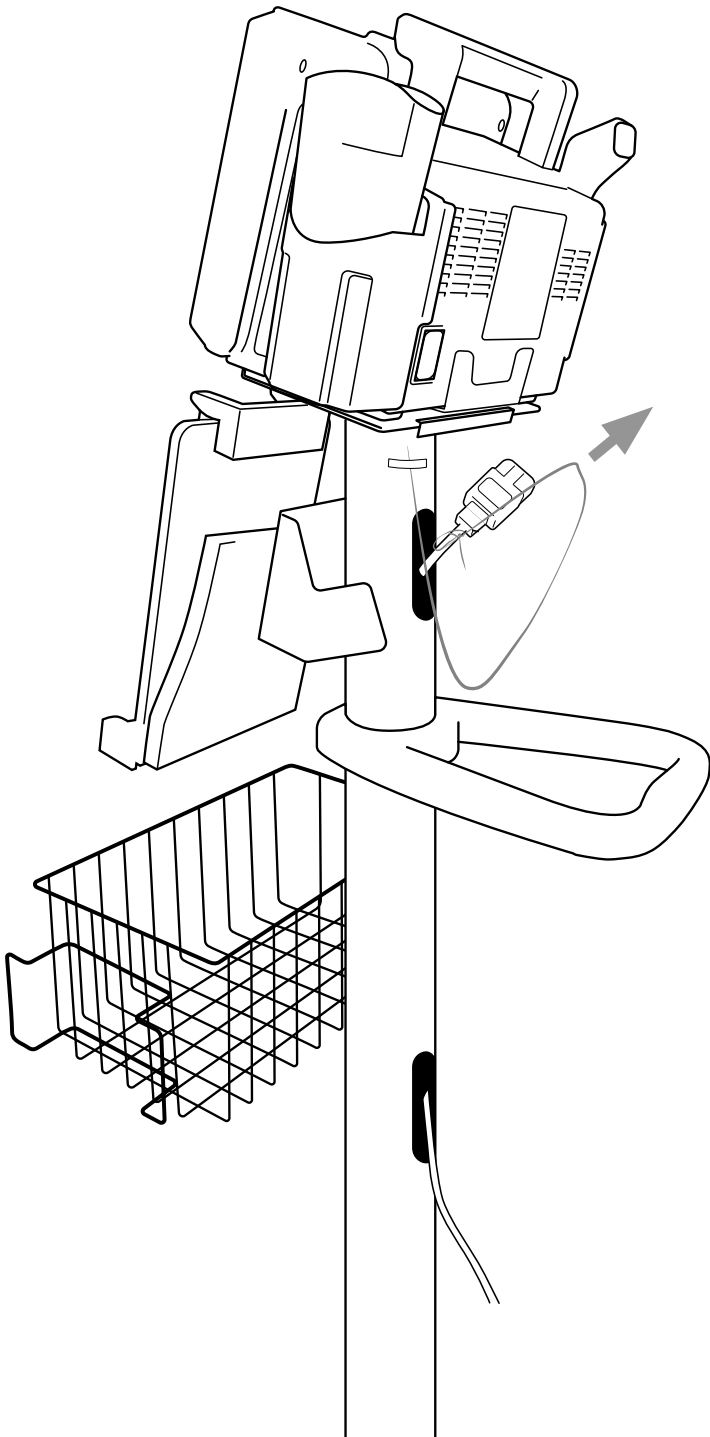

Cord Management *(cont.)*

Connecting Charging Cable to Tablet

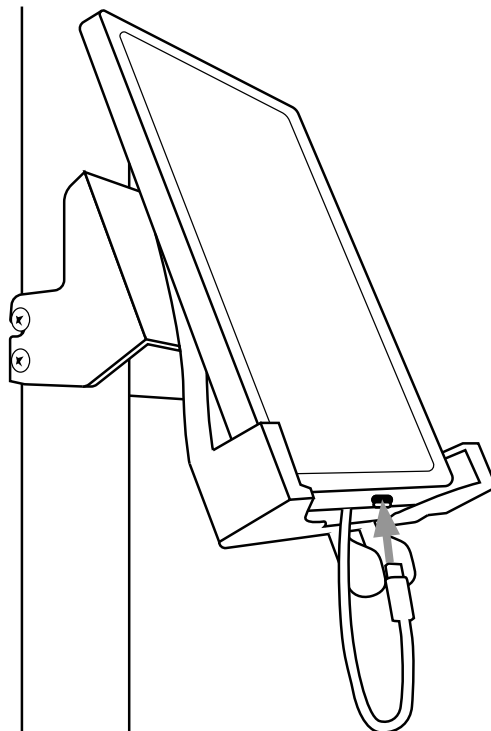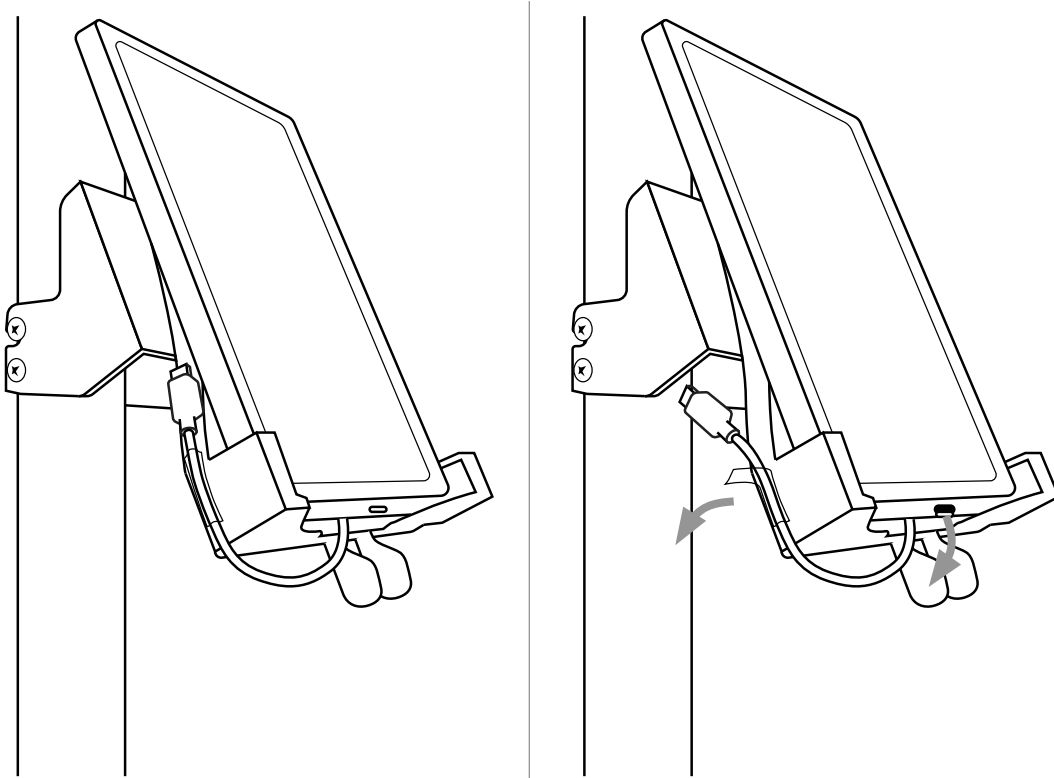

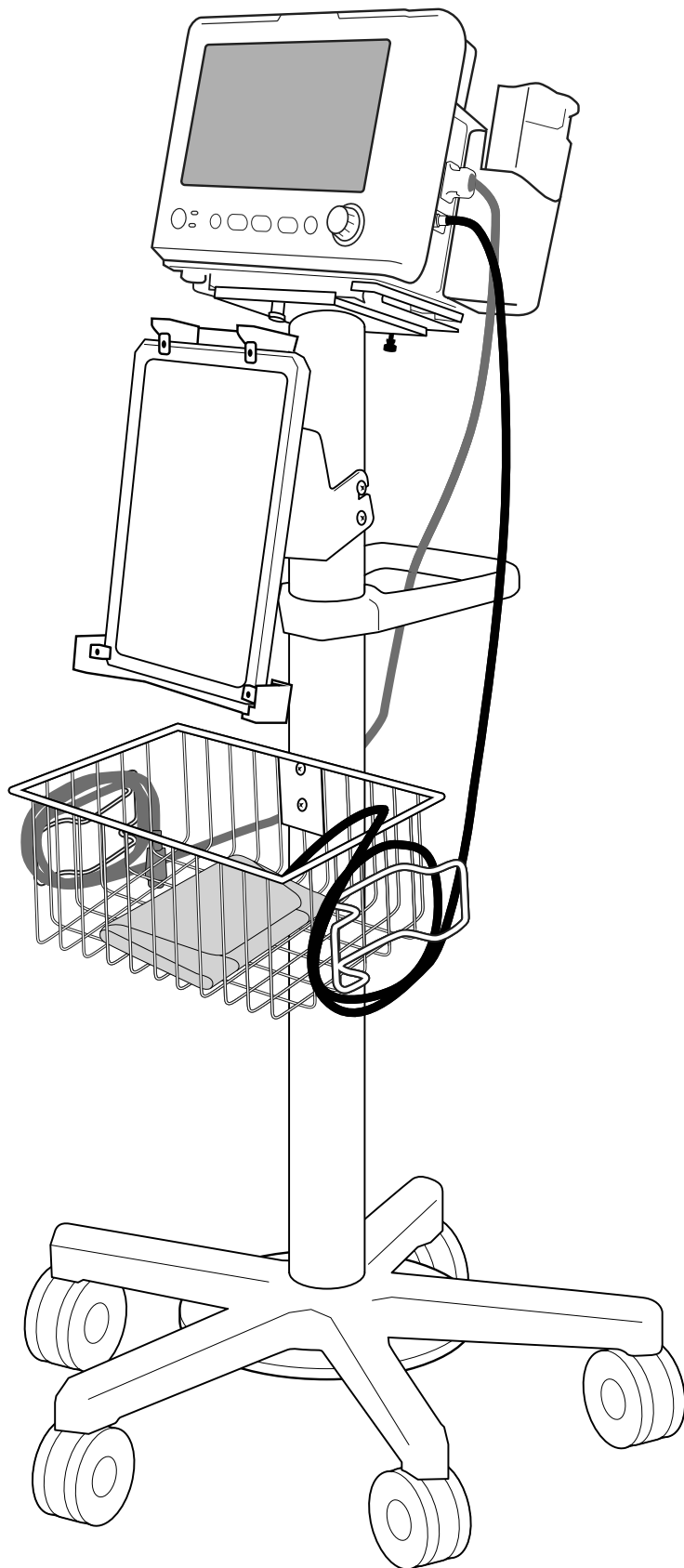

Limited Warranty

We offer to you, as the original purchaser, a warranty for the Attendant® Connected Vital Signs Monitor Cord Management Stand. Our warranty applies for the limited warranty period stated below. If any device or device part listed below is defective in material or workmanship during the applicable limited warranty period, we will repair or replace it at our cost. Please note that the decision to repair or replace a device or device part will be at our discretion. Our warranty applies only if the device is properly maintained by the original purchaser for normal, indoor use and does not cover normal wear and tear, modification of the device, or damage caused by abuse, improper use, failure to maintain, use which exceeds the published device limitations or the combination of any device with another product. In addition, our warranty does not cover fading, colorfastness, stains, spills or exposure to chemicals, odors, heat or light. In certain cases, we may provide you repair or adjustment instructions and/or replacement parts and ask you to perform a repair or adjustment or replace a defective part.

Our warranty gives you specific legal rights, and you may also have other rights, which vary from state to state. Please note that our limited warranty period begins when we ship the device to you. The limited warranty period and our obligations under the warranty end once you transfer the device to someone else, or at the end of the applicable limited warranty period identified below, whichever is earlier.

	Warranty	Anticipated Usable Device Life
Attendant® Connected Vital Signs Monitor Cord Management Stand	5 years	5 years

Anticipated Usable Device Life is based on normal use with proper maintenance, cleaning and storage. You should still inspect, monitor and care for the device as described in this guide, as the device may need to be replaced sooner than anticipated in particular situations.

DIRECT SUPPLY MANUFACTURING, INC. MAKES NO IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE; THESE AND ALL OTHER IMPLIED WARRANTIES ARE SPECIFICALLY DISCLAIMED. TO THE FULLEST EXTENT ALLOWED BY LAW, DIRECT SUPPLY MANUFACTURING, INC. WILL NOT BE LIABLE FOR ANY INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, OR LOST PROFITS THAT MAY RESULT FROM THE DEVICE OR YOUR USE OR INABILITY TO USE THE DEVICE EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. OUR TOTAL LIABILITY TO YOU, IF ANY, IS LIMITED TO THE PRICE OF THE PRODUCT OR SERVICE GIVING RISE TO YOUR CLAIM. Some states do not allow an exclusion or limitation of incidental or consequential damages, or how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. If implied warranties are not excluded, and to the extent allowed by law, we limit any and all implied warranties to the applicable warranty period identified above. Except for rights under any applicable state law, the remedies provided under this warranty are your sole and exclusive remedy for any breach of our warranty and state the entire limit of our responsibilities.



Distributed by Direct Supply® Manufacturing™, Inc. 6635 W Champions Way, Milwaukee WI 53223.

© 2025 Direct Supply Manufacturing, Inc. All Rights Reserved. Direct Supply and all associated logos are the registered trademarks of Direct Supply, Inc., used under license. All other trademarks are the property of their respective owners.