

# Attendant® AVSM3 annual safety checklist

## Simple, reliable maintenance

Your Attendant® AVSM3 monitor is designed for dependable, daily use with **no calibration required**. Instead, a simple **annual safety check** ensures the unit stays compliant and performs reliably.

## Annual safety checklist

Use this checklist during your annual review. Document completion and store it with your records.



### Visual inspection

- Confirm the monitor housing is intact and free of cracks
- Ensure all accessories are present and not worn or damaged
- Replace any worn or missing accessories



### Functional verification

- Power on the monitor and verify that it responds normally
- Replace batteries if needed
- Note any unusual behavior or delays during operation



### Parameter verification

- Confirm the device reads within expected performance ranges
- If the unit falls outside normal parameters, contact the manufacturer



### Remove unsafe equipment

- Any unit identified as damaged, unreliable, or unsafe should be taken out of service immediately



## Questions or support?

If you have any concerns during your check or need additional assistance, please reach out to your **Direct Supply® Customer Success Manager**.